

123 PREMIUM INJECTOR


LOADING GUIDELINES

► 123 Premium preloaded system for easy 2.2mm injection


The Micro+ 123 and the MicroPure 123 lenses are quickly prepared for injection through 2.2mm incisions. The lenses are delivered preloaded in a micro-incision cartridge, which is to be clipped to the 123 Premium single-use injector. This system which requires no lens handling ensures perfect control of asepsis and makes lens injection comfortable and reproducible.

► Secure injection system

The new 123 Premium injector features a safety lock which allows to correctly position the implant for injection. The implantation is performed without risk of haptic catch and release of the plunger.

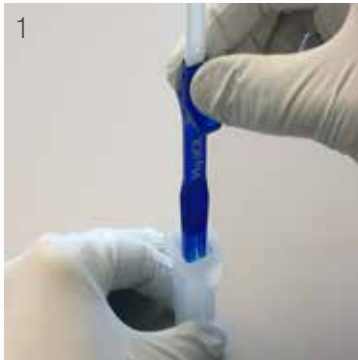


1
Push the injector vertically onto the preloaded cartridge until you hear the "clip".

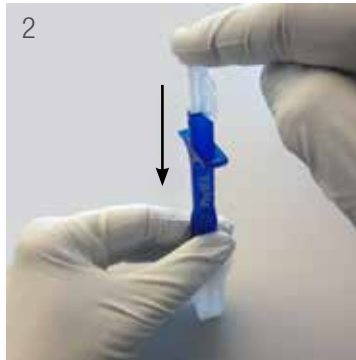

2
Push the plunger down until it abuts the safety catch and ...




3
... keep the plunger in this position for about 3 seconds.

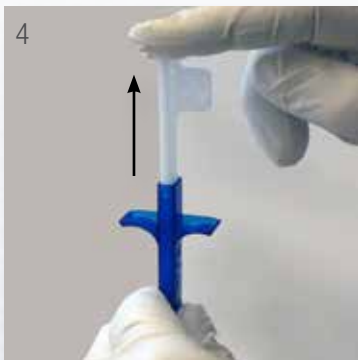

4
Gently release the plunger.



5
Remove the safety catch by a twist motion.

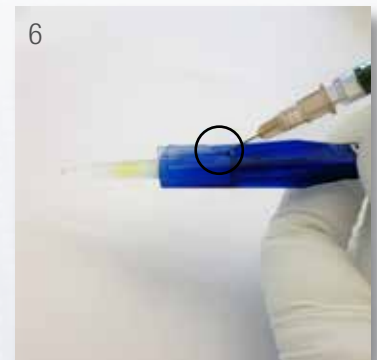

6
Rinse the IOL with BSS by introducing the canula of the BSS syringe into the small hole on the body of the injector.

The assembling of the injector on the cartridge must be performed right before implantation of the IOL.